

**Substances that elicit positive and false-positive results using
Siemens EMIT® II Plus reagents**

Emit® II Plus Opiate Assay (300 ng cutoff) *Calibrated with Morphine at 300 ng/ml*

Drug	ng/ml
6-acetylmorphine	435
Morphine	300
Morphine-3-Glucuronide	626
Codeine	102-306
Dihydrocodeine	291
Hydrocodone	247
Hydromorphone	498
Levallorphan	>5000
Ethylmorphine	240
Levorphanol	1048
Meperidine	>15,000
Nalorphine	5540
Naloxone	360,000
Oxycodone	1500
Oxymorphone	9300

Known False Positives: Ofloxacin and Levofloxacin